



Instructions for using the diagnostic strips


It is essential to use the diagnostic strips in the morning, before going to the toilet for the first time and before washing.

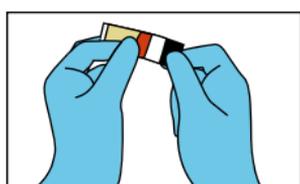

First, remove the black-marked protective film. To do this, hold the red and white-marked strips firmly.

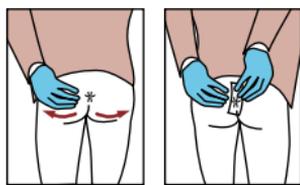

Spread the buttocks. Press the exposed adhesive surface of the red-marked diagnostic strip several times onto the skin of the anal region.

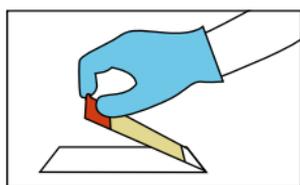

After taking the sample, affix the diagnostic strip, adhesive side down, onto a microscope slide. The white film will now be on top.

Positive test result?

Don't panic, this is not uncommon! In addition to taking the medication prescribed by your doctor for the worm infestation, proper hygiene is important to prevent re-infection.

- Hands must be washed thoroughly after every time you go to the toilet.
- Instruct children to wash their hands regularly at school or nursery as well.
- Thorough hygiene in the anal area is essential.

You can find further hygiene tips for the home and other helpful information in our patient guide on pinworms. Please ask your doctor or pharmacist for a copy.

